

Generic Hana Table

Installation:

- 1 - Large adaptor on the other side, lift on highest position
Small adaptor under the arm rest
- 2 - 2 Grippers are connected after draping.
- 3 - One to the 2 fixation points of the large Hana adapter and the other one to the superior fixation point of the large one and the small one under the arm rest.




Approach:

Superior: number 1 / 6 ME retractor connected to Gripper

Inferior: number 2 / 6 ME retractor connected to Gripper

Lateral: number 3 ME retractor hold by assistant / PA

Anterior: number 4 ME retractor from connected to Gripper

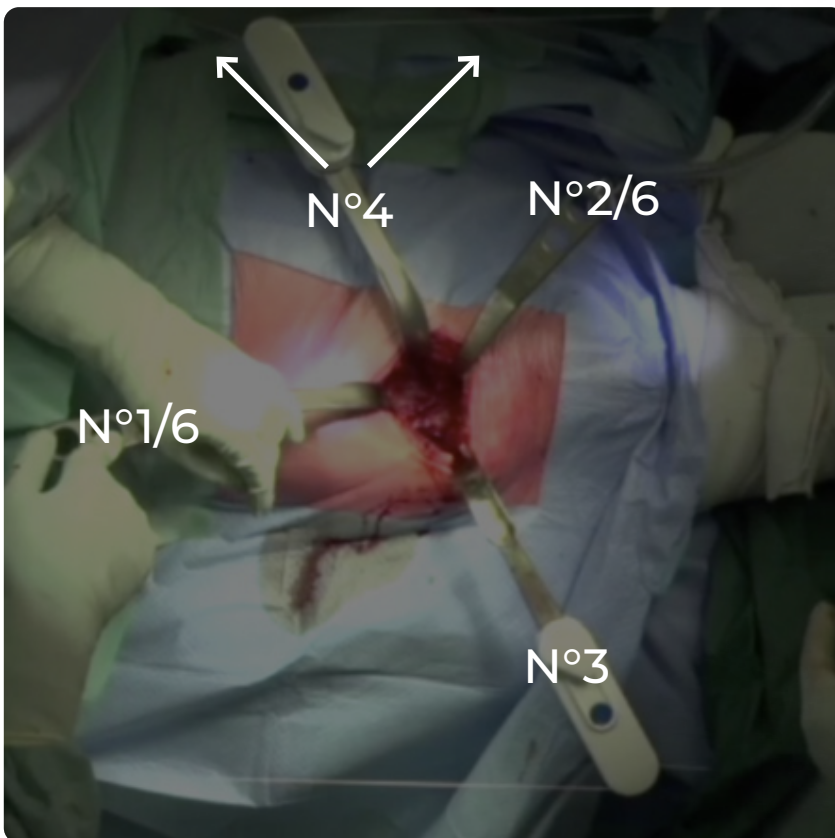


Acetabular exposure:

Anterior: number 4 ME retractor connected to Gripper stays in the same place

Posterior: number 5 ME retractor hold by assistant / PA

Inferior: number 1 ME retractor connected to Gripper

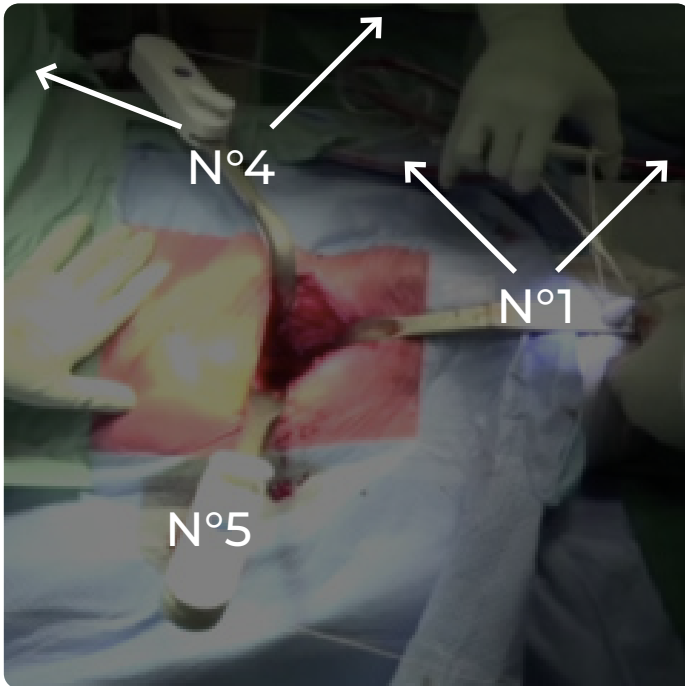

Femoral exposure:

Femoral elevator: number 5 ME retractor connected to the Gripper

Medial retractor: number 1 or 3 ME retractor connected to the Gripper

Lateral retractor: optional Langenbeck or number 1

